Neu ab:
New as of:
Nouveau depuis:
Nuevo a partir de:
Nuovo a partire da:

11.2015

Sirona.
The Dental Company

inLab MC X5, CEREC MC / MC X / MC XL Premium Package

Gebrauchsanweisung für Absaugungen
Operating Instructions for Suction devices
Notice d'utilisation pour machines d'aspiration
Instrucciones de uso para sistemas de aspiración
Istruzioni d'uso per sistemi di aspirazione

Deutsch / English / Français / Español / Italiano

Neu ab:

11.2015

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New as of:

11.2015

Suction devices for inLab MC X5 and CEREC MC / MC X / MC XL Premium Package

Operating Instructions

English

Table of contents

1	General data.....	4
1.1	Dear Customer,.....	4
1.2	Contact information	4
1.3	General conventions and structure of the document	5
1.3.1	Structure of the document.....	5
1.3.1.1	Identification of danger levels.....	5
1.3.1.2	Formats and symbols used	5
1.3.2	Notes to the repository	5
1.4	Warranty and liability.....	6
1.5	Intended use	6
1.6	Legend	6
2	Safety information	8
2.1	Basic safety information	8
2.1.1	Prerequisites	8
2.1.2	Operating	8
2.1.3	Maintenance and repair	8
2.1.4	Modifications to the product	8
2.1.5	Accessories.....	9
2.1.6	In case of damage.....	9
2.2	Connecting the unit	9
2.3	Ventilation slots	9
3	General information on the device	10
3.1	Standards/ approvals/ certifications	10
3.2	Scope of supply.....	11
3.3	Technical description (component and interfaces).....	12
3.4	Technical data	13
3.4.1	InLab MC X5 suction device	13
3.4.2	CEREC MC/MC X/MC XL Premium Package suction device	14
4	Transportation and installation	15
4.1	Transport and unpacking	15
4.2	Disposal of packaging materials	15
4.3	Installation conditions.....	15
4.3.1	Installation site requirements	15
4.4	Connecting the suction device	16
4.4.1	Connecting the suction tube.....	16

4.4.2	Connecting the power cord	17
4.4.3	Connecting the interface cable (for automatic mode).....	17
5	Commissioning	18
5.1	Putting the unit into operation.....	18
5.1.1	Automatic mode	18
5.1.2	Manual operation.....	18
5.1.3	Setting the suction power	18
5.2	Repacking	18
6	Maintenance and repair.....	19
6.1	Changing filter bags and HEPA filters	19
6.2	Changing the suction motor	20
7	Malfunctions	21
7.1	Error messages	21
7.2	Troubleshooting and repair	22
8	Disposal.....	23
9	Spare parts and consumables.....	24

1 General data

1.1 Dear Customer,

Thank you for purchasing this suction device from Sirona.

The suction systems were specially developed for our products inLab MC X5 and CEREC MC / MC X / MC XL Premium Package and offer you maximum safety and the best working conditions when using a Sirona system.

Improper handling and unintended use may lead to hazards and damage. This is why we ask that you read through the operating manual and adhere to it closely. Also pay attention to the safety instructions to prevent personal injury and material damage.

Please read this document completely and follow the instructions exactly. You should always keep it within reach. The original language of this document is German.

Your
inLab and CEREC Team

1.2 Contact information

Customer service center

In the event of technical queries, please use our online contact form at www.sirona.com. In the navigation bar, go to the menu commands "*CONTACT*" / "*Customer Service Center*" and then click the "*CONTACT FORM FOR TECHNICAL QUESTIONS*" button.

Manufacturer's address

Sirona Dental Systems GmbH
Fabrikstrasse 31
64625 Bensheim
Germany
Phone: +49 (0) 6251/16-0
Fax: +49 (0) 6251/16-2591
e-mail: contact@sirona.com
www.sirona.com

1.3 General conventions and structure of the document

1.3.1 Structure of the document

1.3.1.1 Identification of danger levels

To prevent personal injury and material damage, please observe the warning and safety information provided in these operating instructions. Such information is highlighted as follows:

CAUTION

A possibly dangerous situation that could result in slight bodily injury.

NOTICE

A possibly harmful situation which could lead to damage of the product or an object in its environment.

IMPORTANT

Application instructions and other important information.

Tip: Information on making work easier.

1.3.1.2 Formats and symbols used

The formats and symbols used in this document have the following meaning:

 Prerequisite 1. First action step 2. Second action step or ➤ Alternative action ↳ Result ➤ Individual action step	Requests you to do something.
See "Formats and symbols used [→ 5]"	Identifies a reference to another text passage and specifies its page number.
● List	Designates a list.
"Command / menu item"	Indicates commands, menu items or quotations.

1.3.2 Notes to the repository

It is mandatory to keep this operating manual in an easily accessible place for the purpose of later reference. In the event of a sale or transfer of the device to another user, make sure that the device is supplied along with the operating manual, so that the new owner can get acquainted with the operation and the appropriate precautions and warnings

1.4 Warranty and liability

Maintenance

In the interest of the safety and health of users or third parties, it is necessary that maintenance work is carried out at fixed time intervals to ensure the operational safety and reliability of your product.

The operator must ensure the implementation of the maintenance work.

As a manufacturer of electro-medical equipment, we can consider ourselves responsible for the safety characteristics of the device only if maintenance and repairs are carried out only by us or by companies authorized explicitly by us for this purpose and if components are replaced with original spare parts in case of failure.

Exclusion of liability

If the operator does not meet the obligation to carry out such maintenance or fault messages are ignored, Sirona Dental Systems or its authorized dealer does not assume any liability for damage caused.

1.5 Intended use

The suction system is used to extract dry, non-explosive dusts and chips that emerge when processing various materials for the production of dental aids and technologies and dentures. The suction system may only be operated indoors. The suction system must not be used for any other purpose.

This unit must not be used for any other purpose. If the unit is used for any purpose other than the one mentioned above, it may be damaged.

Intended use also includes compliance with these Operating Instructions and the relevant maintenance instructions.

⚠ CAUTION


Follow the instructions

If the instructions for operating the unit described in this document are not observed, the intended protection of the user may be impaired.

1.6 Legend

Year of manufacture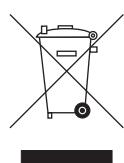

Product disposal symbol (see "Disposal [→ 23]").

Accompanying documents

This symbol can be found on the rating plate on the unit.

Meaning: Observe the Operating Instructions when operating the unit.

Symbols on the packaging

Take note of the following symbols on the packaging:

Top

Protect from moisture


Fragile; handle with care


Stack limit


Temperature during storage and transport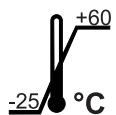

Relative humidity during storage and transport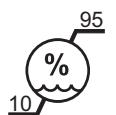

Air pressure during storage and transport

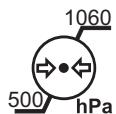2 Safety information

2.1 Basic safety information

2.1.1 Prerequisites

NOTICE

Important information on building installation

The building installation must be performed by a qualified expert in compliance with the national regulations. DIN VDE 0100-710 applies in Germany.

NOTICE

Restrictions regarding installation site

The system is not intended for operation in areas subject to explosion hazards.

NOTICE

Do not damage the unit!

The unit can be damaged if opened improperly.

It is expressly prohibited to open the unit with tools!

2.1.2 Operating

The suction system must only be operated with an integrated filter bag and HEPA filter. Only the approved filter bag and HEPA filter may be used (see "Spare parts and consumables [→ 24]").

2.1.3 Maintenance and repair

As manufacturers of dental instruments and laboratory equipment, we can assume responsibility for the safety properties of the unit only if the following points are observed:

- The maintenance and repair of this unit may be performed only by Sirona or by agencies authorized by Sirona.
- Components which have failed and influence the safety of the unit must be replaced with original (OEM) spare parts.

Please request a certificate whenever you have such work performed. It should include:

- The type and scope of work.
- Any changes made in the rated parameters or working range.
- Date, name of company and signature.

2.1.4 Modifications to the product

Modifications to this product which may affect the safety of the operator, patients or third parties are prohibited by law!

2.1.5 Accessories

In order to ensure product safety, this device may be operated only with original Sirona accessories or third-party accessories expressly approved by Sirona. In particular, only the power cable supplied with the unit or the corresponding original spare part may be used with the unit. The user assumes the risk of using non-approved accessories.

2.1.6 In case of damage

In case of noticeable malfunctions or damage, stop using the instrument immediately and notify your authorized dealer or the manufacturer.

2.2 Connecting the unit

The connection must be established in accordance with the directions provided in these operating instructions. Also observe the information in the Operating Instructions for inLab MC X5, CEREC MC / MC X or CEREC MC XL Premium Package units.

If any devices not approved by Sirona are connected, they must comply with the applicable standards:

- EN 60 950 for information technology equipment, and
- EN 61 010-1 for laboratory equipment.

2.3 Ventilation slots

Under no circumstances may the ventilation slots on the unit be covered, since otherwise the air circulation will be obstructed. This can cause the unit to overheat.

3 General information on the device

3.1 Standards/ approvals/ certifications

CE mark

This product bears the CE mark in accordance with the provisions of directives 2006/95/EC (Low Voltage Directive), 2006/42/EC (Machinery Directive), and 2004/108/EC (EMC Directive). As such, the following standards apply: CISPR 14-1 + A1 + A2, CISPR 14-2, EN55014-1 + A2, EN55014-2, EN61000-3-2, EN61000-3-3, EN60335-1, and EN60335-2-65.

CAUTION

CE mark for connected products

Further products which are connected to this unit must also bear the CE mark. These products must be tested according to the applicable standards.

3.2 Scope of supply

inLab MC X5 230V (REF 6385129) suction device:

- 230 V suction device
- Filter bag (installed)
- HEPA filter (installed)
- Suction tube, complete
- Interface cable
- Operating Instructions
- Power cable (pre-assembled separately)

inLab MC X5 120V (REF 6485929) suction device:

- 120 V suction device
- Filter bag (installed)
- HEPA filter (installed)
- Suction tube, complete
- Interface cable
- Operating Instructions
- Power cable (pre-assembled separately)

CEREC MC/MC X/MC XL Premium Package 230 V (REF 6569730) suction device:


- 230 V suction device
- Filter bag (installed)
- HEPA filter (installed)
- Suction tube, complete (assembled separately)
- Interface cable
- Operating Instructions
- Power cable (pre-assembled separately)

CEREC MC/MC X/MC XL Premium Package 120 V (REF 6569748) suction device:

- 120 V suction device
- Filter bag (installed)
- HEPA filter (installed)
- Suction tube, complete (assembled separately)
- Interface cable
- Operating Instructions
- Power cable (pre-assembled separately)

3.3 Technical description (component and interfaces)

The suction devices are intended for operation on the inLab MC X5 or on the CEREC MC/MC X/MC XL Premium Package milling units. The suction system is equipped with a special filter bag and a HEPA micro filter of filter category H12, which filters fine dust up to 99.5% and is classified as fine dust category M.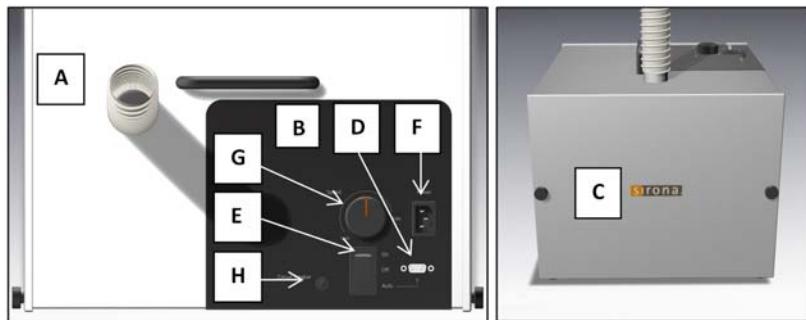

A	Socket for suction tube	E	On/Off switch
B	Control panel	F	Socket for power cable
C	Maintenance flap	G	Suction power control
D	Socket for interface cable	H	Device safety switch

Check the device for any transport damage immediately after unpacking. If there is transport damage, raise this with your supplier immediately.

3.4 Technical data

3.4.1 InLab MC X5 suction device

Type designation	Suction system for inLab MC X5	
Rated line voltage	120V ~	230V ~
Rated power frequency	60 Hz	50 / 60Hz
Nominal current	6.4A	3.6A
Nominal power output	approx. 1000W	
Suction power	min. 120 m ³ /h	
Volume	< 60 dB(A)	
Bag capacity	23l	
Tube diameter	38mm	
HEPA filter	Filter category H12, dust category M, as per EN 60335-2	
Permissible line voltage fluctuations	±10% of nominal voltage	
Type of protection against electric shock	Class 1 equipment	
Overvoltage category	II	
Transport and storage conditions:	Temperature: (-13°F–140°F)	-25°C – +60°C
	Relative humidity: Air pressure:	10%–95% 500 - 1060 hPa
Operating conditions	Ambient temperature: Pollution degree: Air pressure: Installation location: Relative humidity:	5°C – 40°C 2 acc. to IEC 60664-1 700 - 1060 hPa Indoors, ≤ 3,000 m above sea level max. 80% at 31°C (87.8°F) decreasing linearly to max. 50% at 40°C (104°F)
Operating mode	Continuous operation	
Dimensions (WxHxD) in mm	350 x 510 x 360	
Approx. weight	16kg	

3.4.2 CEREC MC/MC X/MC XL Premium Package suction device

Type designation	Suction device for CEREC MC/MC X/MC XL Premium Package	
Rated line voltage	120V ~	230V ~
Rated power frequency	60 Hz	50 / 60Hz
Nominal current	6.4A	3.6A
Nominal power output	approx. 1000W	
Suction power	min. 120 m³/h	
Volume	< 60 dB(A)	
Bag capacity	5l	
Tube diameter	38mm	
HEPA filter	Filter category H12, dust category M, as per EN 60335-2	
Permissible line voltage fluctuations	±10% of nominal voltage	
Type of protection against electric shock	Class 1 equipment	
Overvoltage category	II	
Transport and storage conditions:	Temperature: Relative humidity: Air pressure: Ambient temperature:	-25°C – +60°C (-13°F–140°F) 10%–95% 500 - 1060 hPa 5°C – 40°C (41°F – 104°F)
Operating conditions	Pollution degree: Air pressure: Installation location: Relative humidity:	2 acc. to IEC 60664-1 700 - 1060 hPa Indoors, ≤ 3,000 m above sea level max. 80 % at 31°C (87.8°F) decreasing linearly to max. 50 % at 40°C (104°F)
Operating mode	Continuous operation	
Dimensions (WxHxD) in mm	360 x 340 x 260	
Approx. weight	12.7 kg	

4 Transportation and installation

4.1 Transport and unpacking

All products from Sirona are carefully checked prior to shipment. Please perform an incoming inspection immediately after delivery.

1. Check the delivery note to ensure that the consignment is complete.
2. Check whether the product shows any visible signs of damage.

NOTICE

Damage during transport

If the product was damaged during transport, please contact your carrying agent.

If return shipment is required, please use the original packaging for shipment.

4.2 Disposal of packaging materials

The packaging must be disposed of in compliance with the relevant national regulations. Please observe the regulations applicable in your country.

4.3 Installation conditions

4.3.1 Installation site requirements

The inLab MC X5 suction device requires an even base of approx. 400 mm x 400 mm (W x D).



The CEREC MC/MC X/MC XL Premium Package suction device requires an even base of approx. 360 mm x 260 mm (W x D).

Install the suction device in such a way that the main switch can be pressed easily. To enable simple replacement of the filter and bag, we recommend orientating the maintenance flap to the front with the two knurled nuts. The ventilation slots on the bottom side and rear side of the suction system must be kept unobstructed. The distance between the rear side of the unit and the room wall must be at least 30 mm. If the suction device is integrated in a cabinet, this must be vented sufficiently to prevent the suction device from overheating.

The suction device must not be set up in the immediate vicinity of considerable humidity development.

4.4 Connecting the suction device

4.4.1 Connecting the suction tube

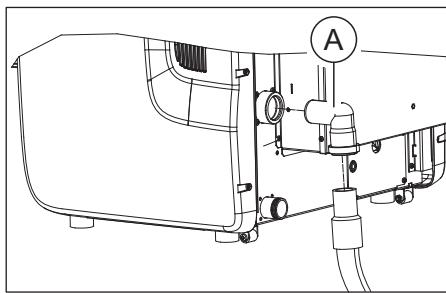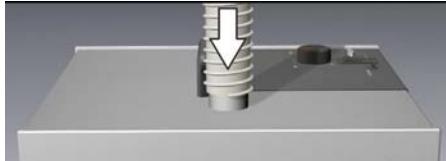1. Connect one end of the suction tube to the available connection point on the suction device. The connection point is located on the top side next to the control panel with the suction power controller.

2. Connect the other end of the tube to the socket (A) on the rear side of the processing machine.

or

- Connect the other end of the tube to the rear side of the milling unit.

Notes on the suction tube:

The suction tube is supplied at a length of approx. 2.40 m (inLab MC X5) or 2.0 m (CEREC MC/MC X/MC XL Premium Package). When connecting the suction device to the unit, please ensure that no sharp bends occur over the full length of the suction tube.

Reduce the length of the tube according to your requirements and your installation location. Note that suction power drops along the length of the tube. You obtain decent suction power if the suction device is placed immediately below the machine and you have a tube length of 1.2 m or less.

4.4.2 Connecting the power cord

NOTICE

Grounded power outlet

The unit must be connected to a grounded power outlet.

1. Insert the power cable into the relevant socket on the suction system.
2. Plug the other end into an appropriate power socket with a protective ground terminal.

Note on the power cable:

The suction system must only be operated with a power cable with a plug system designed for the relevant country.

Check the voltage specification on the rating plate. The system must conform to the country-specific supply voltage.

4.4.3 Connecting the interface cable (for automatic mode)

1. Plug the 9-pole connector into the socket of the suction device.
2. Plug the 15-pole connector into the socket of the processing machine.

5 Commissioning

5.1 Putting the unit into operation

5.1.1 Automatic mode

Note on the automatic mode:

The processing machine monitors the correct socket (interface cable and suction tube) and the operation of the suction system during the running processes.

5.1.2 Manual operation

NOTICE

Manual mode only permitted for inLab MC X5. For CEREC MC/MC X/MC XL Premium Package only automatic mode is supported.

1. Prior to commencing processing, set the on/off switch to the *On* position.
2. Once processing has been completed, set the on/off switch to the *Off* position.

5.1.3 Setting the suction power

inLab MC X5

- Set the suction power to the maximum (*max*).

CEREC MC/MC X/MC XL Premium Package

- Set the suction power to the minimum (*min*). If required, the suction power can be set to the maximum.

5.2 Repacking

1. Set the on/off switch to the position *OFF*.
2. Remove the power plug, interface cable, and suction tube.
3. Check the suction device for completeness according to the scope of supply.
4. Pack the device safely, ideally in the original transport packaging.

6 Maintenance and repair

CAUTION

Risk of uncontrolled release of fine dust

If the suction device is not operated with the inserted filter bag or HEPA filter provided, fine dust may be discharged uncontrollably.

- Only operate the suction system with original spare parts (see "Spare parts and consumables [→ 24]").

6.1 Changing filter bags and HEPA filters

Changing filter bags

If there is a significant fall in suction power, the filter bag could be full and will need replacing.

NOTICE

Do not jam the filter bag

Make sure that the cap is sealed properly and the filter bag is not jammed.

Changing HEPA filters

If suction power is still weak after replacing the filter bag, then replace the HEPA filter, too. The HEPA filter is located behind the filter bag.

1. Loosen the two knurled nuts on the maintenance cover.
2. Remove the lid.
3. Remove the filter bag from the nozzle.
4. Then release and remove the two Phillips screws on the perforated sheet on the inside of the suction system.
5. Take the perforated sheet out and remove the HEPA filter.
6. Put a new HEPA filter on.
7. Put the perforated sheet back on and screw it down with the two Phillips screws.
8. Put the filter bag back on again.
9. Put the maintenance cover on and screw it down with the two knurled nuts.

NOTICE

Do not jam the filter bag

Make sure that the cap is sealed properly and the filter bag is not jammed.

6.2 Changing the suction motor


The motor of the suction device has a service life of approx. 1,200 hrs. You will notice the motor is worn when suction does not start in either automatic or manual mode. When this happens, have the motor replaced by a trained service engineer.


7 Malfunctions

7.1 Error messages

Error	Possible cause	Remedy:
Suction air not present; suction device works when on/off switch is in the <i>On</i> position.	Suction tube blockage	Check suction tube for open passage; remove blockages.
	Leak on suction tube or socket	Check suction tube for leak tightness and tight connection to the suction unit and processing machine; seal the tube / connect it tightly if required.
	Fault inside the processing machine	Contact services
Software error message: Suction device vacuum switch	Suction power controller is at <i>min</i> on the control panel.	Set suction power controller to <i>max</i> .
	Filter bag / HEPA filter occupied	Replace filter bag Replace HEPA filter
	Suction tube blocked	Check suction tube for open passage; remove blockages if necessary.
	Interface cable defective	Replace interface cable
	Fault inside the processing machine	Contact services

7.2 Troubleshooting and repair

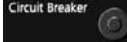Symptom	Possible cause	Remedy:
Suction device does not start up, either in automatic mode or with the on/off button set to <i>On</i> .	Power cable not connected.	Check power cable for correct connection.
	Voltage supply inactive.	Check voltage source
	The fuse on the suction device has tripped.	 Press the device safety switch, see Technical description (component and interfaces) [→ 12]
	Suction motor superheated.	Let the suction system cool down; ensure there is sufficient air circulation, especially around the ventilation slots.
	Suction motor defective / worn out.	Contact services
Insufficient suction power	Suction power controller is at <i>min</i> on the control panel.	Set suction power controller to <i>max</i> .
	Filter bag is full.	Replace filter bag
	HEPA filter occupied.	Replace HEPA filter
	Suction tube blocked.	Check suction tube for open passage; remove blockages if necessary.
	Leak on suction tube or socket.	Check suction tube for leak tightness and tight connection to the suction unit and processing machine; seal the tube / connect it tightly if required.
Very strong suction power	Filter bag / HEPA filter not seated correctly.	Open maintenance cover. Check filter bag and HEPA filter are seated correctly.
Suction very loud	Turbine touches the housing.	Contact services

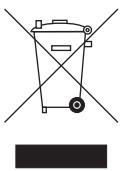8 Disposal

In accordance with Directive 2012/19/EU and national disposal regulations regarding old electrical and electronic devices, please be advised that such items must be disposed of in a special way within the European Union (EU). These regulations require environmental friendly usage/disposal of old electrical and electronic devices. Such items must not be disposed of as domestic refuse. This has been expressed using the icon of the “crossed out trash can” since March 24, 2006, amongst other methods.

Disposal procedure

We feel responsible for our products from the first idea to their disposal. For this reason, we give you an option to return our old electronic and electrical devices.

If you wish to dispose of your devices, please proceed as follows:

In Germany

To initiate return of the electrical device, please send a disposal request to enretec GmbH. You have the following options here:

- Use the “Returning an electrical device” button under the “eom” menu item on the enretec GmbH homepage (www.enretec.de).
- Alternatively, you can also contact enretec GmbH directly.

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Tel.: +49 3304 3919-500
E-Mail: eom@enretec.de

In accordance with the national disposal regulations regarding old electrical and electronic devices (ElektroG), as the manufacturer, we assume the costs for disposing of the electrical and electronic devices in question. Disassembly, transport and packaging costs shall be borne by the owner/ operator.

Prior to disassembly / disposal of the product, it must be fully prepared (cleaned / disinfected / sterilized).

If your unit is not permanently installed, it will be collected from the practice. If it is permanently installed, it will be picked up curbside at your address by appointment.

Other countries

For country-specific information on disposal, contact your local dental dealers.

9 Spare parts and consumables

REF	Designation
6385244	Filter bag, double pack (inLab MC X5)
6578095	Filter bag, double pack (CEREC)
6385277	HEPA filter
6384841	Suction motor, 230 V
6384791	Suction motor, 120V
6453851	Suction tube, complete, spare (inLab MC X5)
6564590	Suction tube, complete (CEREC)
6453869	Interface cable, spare
5864553	Power cable (EU)
5864520	Power cable (GB)
5864546	Power cable (IT)
5864538	Power cable (CH)
5864512	Power cable (US)
5865147	Power cable (JP)
5966119	Power cable (AU/NZ)
6412766	Power cable (BR)

We reserve the right to make any alterations which may be required due to technical improvements.

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© Sirona Dental Systems GmbH 2015