# Dental zirconia blank for aesthetic restoration

- ⓔ Bloque de circona dental para restauraciones estéticas
- ⓕ Ébauche de zircone dentaire destinée à la restauration esthétique
- ⓓ Dentaler Zirkonrohling für die ästhetische Restauration
- ⓔ Blocco di zirconia dentale per restauro estetico
- Ⓝ Tandheelkundige zirconium blank voor esthetische restauratie
- Ⓟ Bloco de zircónia dentária para restauração estética
- Ⓢ Dentální zirkonový polotovar pro estetickou náhradu
- Ⓟ Półfabrykat z cyrkonu dentystycznego do odbudowy estetycznej
- ⓓK Dental zirconia blank for aesthetic restoration
- ⓔ Dental Zirconia Blank för estetisk restaurering
- ⓃO Blokk av dentalt zirkoniumoksid for estetisk restaurering

Medical Device

Date of  
manufacture

Use-by date

Non-Sterile

Do not  
reuse

Batch code

HENRY SCHEIN INC.  
135 DURYEA ROAD  
Melville, NY 11747 USA

HENRY SCHEIN SERVICES GmbH  
MONZA STRASSE 2A  
D-63225 LANGEN, GERMANY  
WWW.HENRYSCHEINBRAND.COM  
WWW.ZIRLUX.COM  
REV.2022/01

**Instruction for use**

This product should be handled by professional dental technician, sit and adjust by dental specialist and follow doctor's instruction.

**Product Name:** Dental zirconia blank for aesthetic restoration

**Intended use:**

The product is mainly used in prosthetic treatment, When the flexural strength of Dental zirconia blank for aesthetic restoration after sintering is between 300-500MPa, the product can be used for veneering, inlay, single crown, and substructure ceramic for three-unit prostheses not involving molar restoration, When the flexural strength of Dental zirconia blank for aesthetic restoration after sintering is over 500 MPa, it can be used for veneering, inlay, single crown, and substrate ceramic for three-unit prostheses. All blanks are processed through dental laboratories or by dental professionals.

**Chemical compositions:**

- ZrO<sub>2</sub>+HfO<sub>2</sub>: 86.3%~94.2%
- Y<sub>2</sub>O<sub>3</sub>: 5.8%~9.7%
- Fe<sub>2</sub>O<sub>3</sub>: <0.5%
- Al<sub>2</sub>O<sub>3</sub>: <0.5%
- Er<sub>2</sub>O<sub>3</sub>: <2%
- Other oxide: <0.5%
- CTE (25-500°C) : (10.5±0.5) ×10<sup>-6</sup>·K<sup>-1</sup>

**Instruction for use:**

1. Fix the zirconia blank in the specific clamping device.
2. Start the milling process.
3. Remove milled framework from blank and pre-trim with diamond disc.
4. Remove dust from the framework with air gun.
5. Color frame with coloring liquid, Final-dry frame under infrared lamp (See manufacturer manual of colouring liquid).
6. Load into furnace.
7. Select sintering program and start sintering (see sintering schedule).
8. Remove sintered framework from furnace after the furnace temp drop to room temperature.
9. Check frame on working model for sitting and margin fit. Trim with grinder and polisher if required.
10. Blast using dia-50 micron Alumina at pressure 2-2.5 bar. Clean frame with brush using water. Do not steam-clean.
11. Frame is now ready for veneering, choose porcelain and glaze to finish the restoration.
12. Now the restoration is ready for the try-in.

**Shelf period:** 5-years is an estimated replacement time. The product is not expected to degrade during this time period.

**Storage:**

1. Keep container dry.
2. Store at room temperature.

**NOTE:** The product shall be used only by trained personnel. The manufacturer is not liable for any erroneous results, since we have no control if this instruction is properly followed or if the product is appropriately used. Any claims for damages therefore shall be limited exclusively to the commercial value of our products.

Please make sure that the dentist has received all of the necessary information for the correct use of the product.

According to the EU Medical Devices Regulation, users / patients are obliged to report serious events with a medical device to the manufacturer and to the competent authority of the country in which they occurred.

**Cautions:** Federal Law restricts this device to sale by or on the order of a dentist.

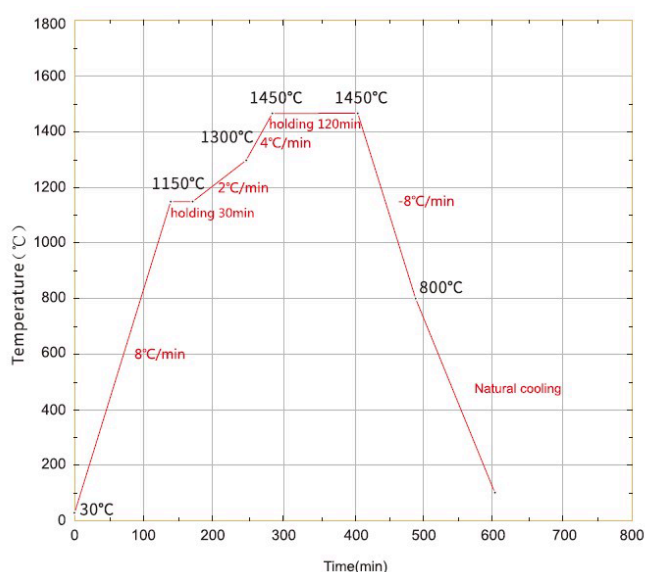
**Contraindications:** Not found.

**Safety instruction:**


1. Avoid collision with hard material, avoid squeeze or shaking.
2. Don't apply to patient before the material is fully sintered.
3. This product should only be handled by dental specialist.
4. Wear suitable face mask to prevent fine dust inhalation. Avoid fine dust air pollution by using the air blaster in short bursts only rather than a continuous flow.

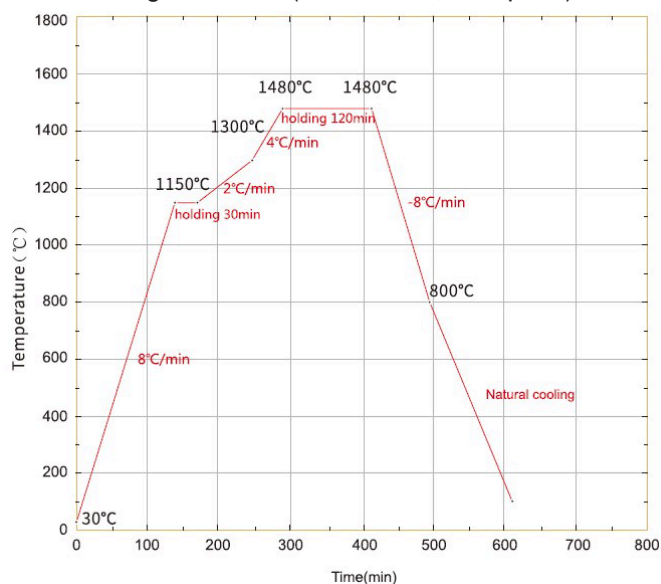**Disposal:** The material is not hazardous to the environment.

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